

PRODUCT INSTRUCTION SHEET

REFERENCE

Belmont PU Corner Shower Stool (Code 124)

DESCRIPTION

Adj. Ht. triangular shower stool which fit neatly into the corner of the shower base allowing full access for the carer. Tough, plastic coated (peel resistant), tubular steel frame. Comfortable PU (polyurethane) moulded padded seat which is resistant to MRSA and other microbiological contaminants.

TECHNICAL SPECIFICATIONS

Seat height Adjustable 51-61cm (20-24")

Seat Width: 400mm (16")

Seat Depth: 335mm (13 1/8")

Footprint: 425 (16 3/4") x 425mm (16 3/4")

Product Weight: 4.5Kg (10 lbs)

Max User Weight: 165Kg (25st)

Height Adjustment

1. Put seat upside down on floor/flat surface.
2. Adjust to the correct height by removing pin clips, sliding extension leg to required position and replace pin clip.
3. Ensure all three leg extensions are secured to the same adjustment hole to ensure even height.

Using the Shower Stool

Place in shower. Ensure all three legs are level and in contact with the shower floor. Sit centrally on the shower stool.

Safety Precautions

Do not exceed the maximum user weight limit.
Inspect at regular intervals for wear and damage.
Do not use if rubber feet are damaged.

Cleaning Instructions

Clean regularly, wiping down with mild detergent /disinfectant solution.

