

Please follow the fixing and care instructions to ensure your product is fitted safely and retains its high quality finish.

**This instruction leaflet applies to the following product:**

AP210122 Bath Bench

**General Safety Advice**

1. The Bath Bench must only be used inside a bath.
2. It must only be used for sitting – DO NOT STAND ON THE BATH BENCH.
3. The maximum load for the Bath Bench is 198lbs/90kg evenly distributed. DO NOT EXCEED THE MAXIMUM LOAD.
4. The Bath Bench is fits most 28inch/70cm wide parallel sided baths. Always ensure that the Bath Bench is firmly supported at both ends before use (see Fig 1).

Fig 1



Bath Bench fitted correctly and securely supported at both ends.

**Product Care Advice**

To retain the best quality finish, clean product regularly with a soft damp cloth. Do not use abrasive or chemical cleaners, as these will damage the product.

**Please retain this leaflet for future reference.**

Croydex Ltd, Central Way, Andover, Hants, SP105AW, UK  
Tel: +44 (0) 1264 365881 ~ Fax: +44 (0) 1264 356437  
website: [www.croydex.com](http://www.croydex.com) ~ Email: [info@croydex.co.uk](mailto:info@croydex.co.uk)