

Assembly instructions for :-

### AP400222 - Prima Modular Shower Stool

Thank you for purchasing this Croydex product. Please follow the assembly and care instructions to ensure your shower stool is assembled safely and retains its high quality finish.

**No tools are required to assemble this product although a blunt instrument may help the removal of the seat caps.**


**A hex head key is supplied to re-tighten bolts on the underside of the seat if necessary.**

#### Safety and Care Advice


1. This Shower Stool must only be used on a flat level surface.
2. It must only be used for sitting – **DO NOT STAND ON THE SHOWER STOOL.**
3. Ensure all parts are assembled correctly and are secure before use.

#### Product Assembly Instructions

1. The tubular components are assembled by depressing the spring-loaded buttons at the ends of the tubes. Care must be taken not to pinch fingers when pressing the spring-loaded buttons. (See Fig.1)
2. Depress the spring-loaded button at the end of the leg sub-assembly and insert into the frame on the underside of the seat. Ensure the spring-loaded button engages fully with the corresponding hole in the tube. (See Fig.2)

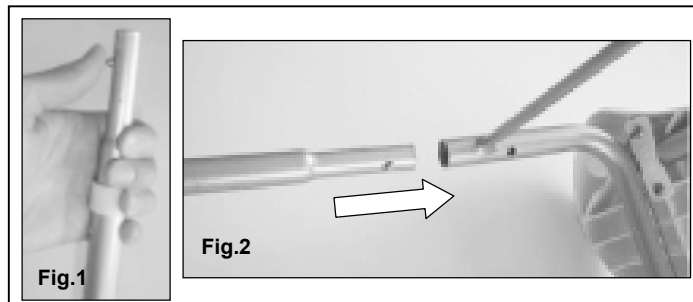


3. To fit the armrests, remove the plastic caps from the seat by releasing the catches securing them to the underside of the seat moulding (See Fig.3), and discard them as illustrated in Fig.4.
4. Remove the plastic retaining clips from the ends of the armrest tubes (See Fig.5) and insert the tube ends into each side of the seat. (See Fig.6) Note the armrests are handed.

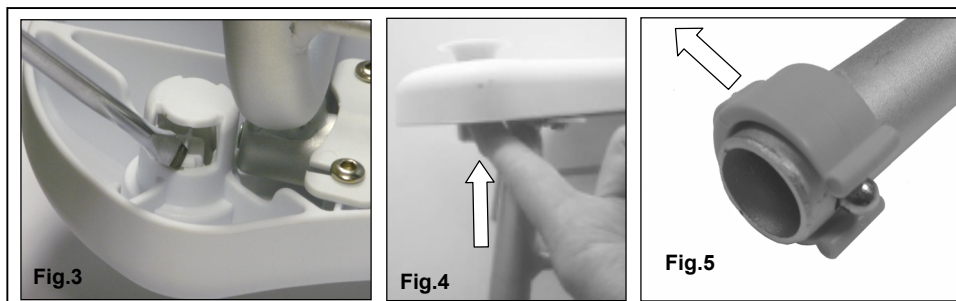

5. Rotate the armrests until they are vertical and align the plastic backrest with the tube ends. Simultaneously depress the spring-loaded buttons on the tube ends and slide through the backrest moulding. Ensure the backrest is in the correct orientation before assembly. (See Figs.7 & 8)
6. Locate the ends of the armrests into the corresponding holes in the top of the seat and push down to clip into position. (See Fig.8)

7. Finally push the two retaining clips over the tube and into the holes on the underside of the seat to lock the armrests in place. It may be necessary to manipulate the armrests to line the holes up before attaching the retaining clips. (See Fig.9)

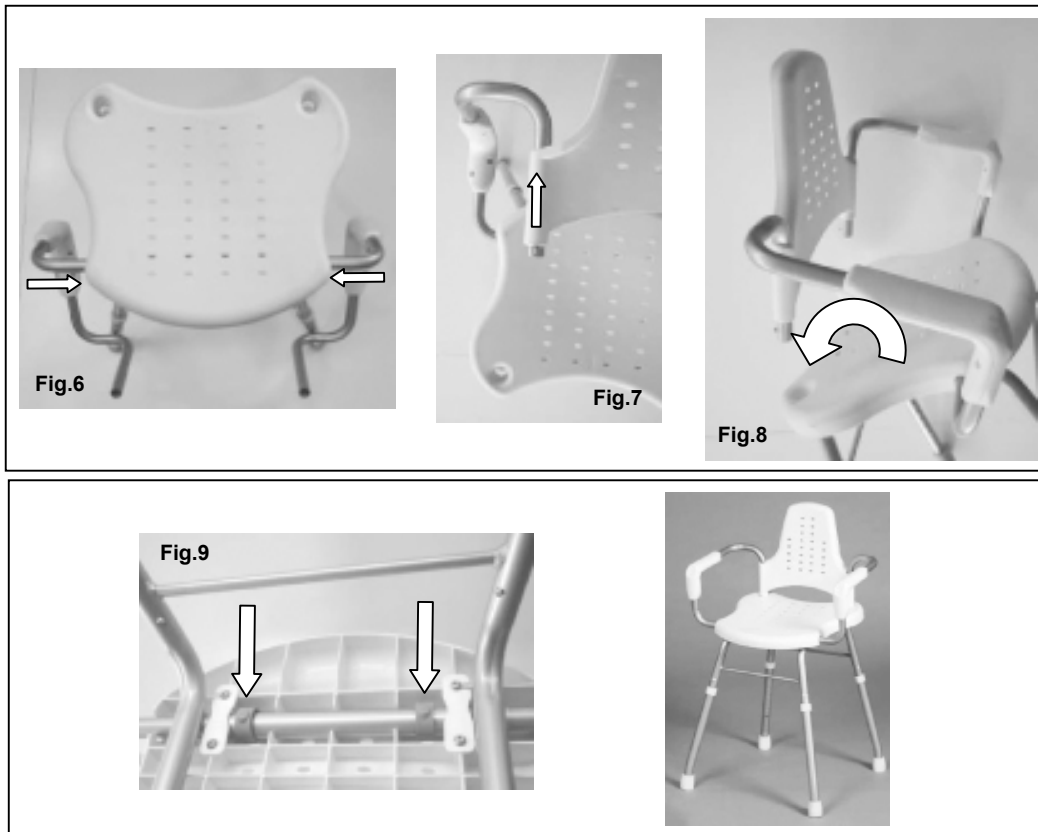

8. Adjust the seat height by pressing the spring-loaded button nearest to the foot and sliding the leg up or down to the desired height and clip into place. Ensure the button is located into the matching hole in each of the four legs.
9. Test for stability before use.
10. The product is designed for use in a static mode only. As with all devices designed to assist the elderly or disabled, care should be taken to avoid sudden or undue movement during use.

#### Additional Information

- Maximum safe working load for the Shower Stool is 420lb (190kg). This load must be evenly distributed across the surface area of the stool.
- Maximum seat height 22" (560mm)
- Minimum seat height 16 ¼" (410mm)

#### Product Care Advice

- To retain the best quality finish, clean product regularly with a soft damp cloth.
- Do not use abrasive or chemical cleaners, as these will damage the product.

#### Markings



BS EN 12182:1999 Technical Aids for Disabled Persons - General Requirements and Test Methods.

**Please retain this leaflet for future reference.**

For details of the other items in the Croydex range please visit our website: [www.croydex.co.uk](http://www.croydex.co.uk)

Croydex Limited, Central Way, Andover, Hants, SP10 5AW, UK  
Tel: +44 (0) 1264 365881. Fax: +44 (0) 1264 356437  
Website: [www.croydex.com](http://www.croydex.com) E-mail: [info@croydex.co.uk](mailto:info@croydex.co.uk).