

Back Rest Rail and Pad Fitting Instructions

BRPAD B BRPAD W

Congratulations on purchasing your back rest rail and pad.

The back rest rail offers a safe supporting hold providing:

- A padded support for the back for use in DocM WC Applications.
- Maximum rated load of 90kg.
- Complies with DocM requirements.
- 35mm Diameter Rail.

Warning: Before fitting, ensure that there are no concealed electrical cables or pipes where the fixing holes are to be drilled. This can be checked using electronic detecting tool. If installing a metallic rail in a bath/shower area the rail may need to be earthed (IEE wiring regulations); not all rails are suitable for use in a wet area.

- 1. Ensure the wall and fixings being used are suitable.


 Bristan are unable to specify fixings due to the number of fixing/wall type combinations.
- 2. Position the Back Rest Rail on the wall in a horizontal position.
- 3. Drill the holes to the correct depth for the wall and fixing type used.
- 4. Ensure the Back Rest Rail is securely fastened to the wall and fit the flange covers to conceal the fixings.


If in doubt about anything on this sheet consult a suitably qualified professional with experience in fitting products of this nature.



Using the Back Rest Rail

- Before first use, check that the Back Rest Rail is securely fixed to the wall.
- Use the Back Rest Rail to assist, NEVER apply full weight to the product.
- Regularly clean the Back Rest Rail with a non-abrasive cleaner.
- All fixings and bolts should be checked on a regular basis by a competent person.

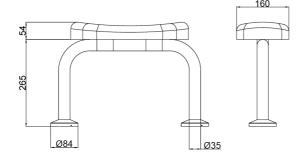

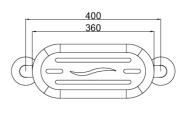

Please Note: Any fixings supplied in this package will not suit every type of wall. Consult a competent person if unsure.

Bristan, Birch Coppice Business Park, Dordon, Tamworth, B78 1SG Telephone: 0844 701 6274 Facsimile: 0844 701 6275 Web Site: www.bristan.com Email: enquire@bristan.com





