

warranty details

Your Drive DeVilbiss branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, contact your Drive DeVilbiss stockist.



Drive DeVilbiss Healthcare Ltd, Heathfield Lane,
Birkenshaw, West Yorkshire, BD11 2HW

Tel: 0845 0600 333 | Fax: 0845 0600 334

Email: info@drivedevilbiss.co.uk

www.drivedevilbiss.co.uk

The word 'drive' in a stylized blue font with a yellow accent.

The DeVilbiss Healthcare logo, featuring a square icon with a white 'D' and the words 'DeVilbiss HEALTHCARE' in a smaller blue font.



Wall Mounted Shower Seat

User Manual

CE

introduction

The Drive DeVilbiss Folding Shower Seat is available in two models. The SWALLOO2 is constructed from aluminium and features height adjustable legs for support.

The SWALLOO1 is constructed from sturdy powder coated steel. Both models feature a HD polyethylene seat with drainage holes and fold away to the wall to save space when not required.

installation

Both models must be permanently fitted to a wall before use. They feature six 8mm diameter holes, and should only be fitted by a competent person and fastened to a wall which has the capacity to withstand the vertical load and leverage applied during use. Fasteners are not supplied because the method of fixing will depend on material and thickness of the wall. The person fitting the seat will be able to supply suitable fasteners.

For the SWALLOO2, set the required leg height before installing the plate to the wall. To set the leg height pull out 'E' clip retaining the leg. Slide the leg to the required position and reinsert the 'E' clip. Repeat for the other leg, ensuring before legs are set to the same height.

cleaning and maintenance

Both models require minimal maintenance. Periodically check the wall fasteners and pivot points are secure.

To clean the seat, occasionally wipe the seat with a soft cloth using a mild household detergent or disinfectant and warm water. Do not use harsh abrasives or bleach as these can cause damage and/or discolouration.

shower seat safety

- The bath seat should only be used as an aid for sitting. Do not stand on or misuse the bath seat.
- Do not exceed the weight limit of the seat.
- Some users and children may require supervision whilst using the bath seat.


specification

Model: SWALLOO1 - **Maximum User Weight 115kg**

Model: SWALLOO2 - **Maximum User Weight 160kg**

On both models, the seat is 48cm (19") wide x 29cm (11") deep and 12cm (5") from the wall.

The height on the SWALLOO2 is 43 - 58cm (17" - 23") to the top of the seat.

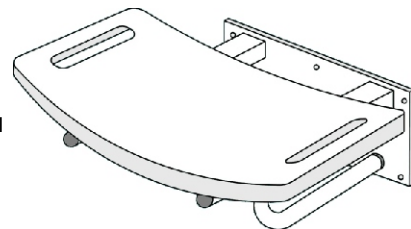

The SWALLOO1 is illustrated right: