
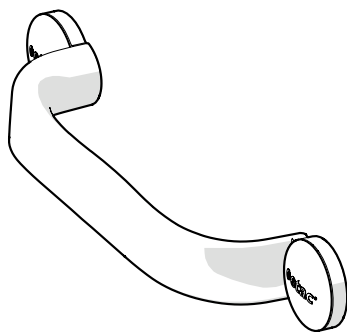
Etac Flex


Manual

etac®

75339.4 23-01-16

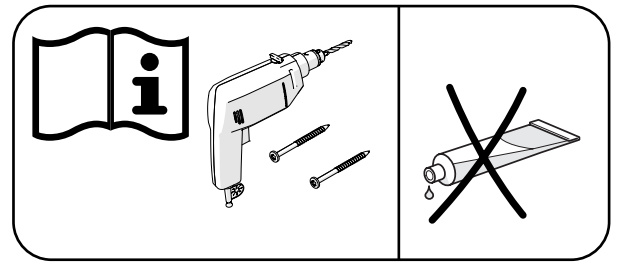30 cm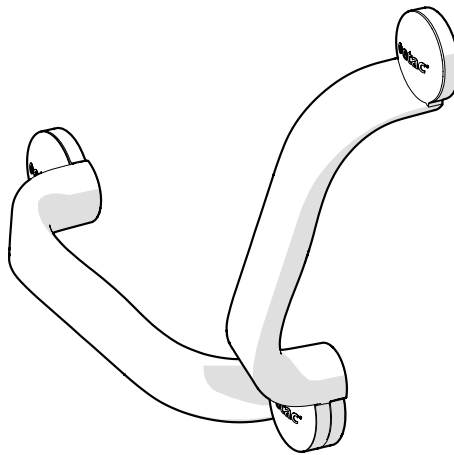


60 cm

Manual


sv Svenska 4	de Deutsch 5-6	it Italiano 7-8
no Norsk 4	fi Suomi 6	es Español 8
da Dansk 4-5	fr Français 6-7	pt Português 8-9
en English 5	nl Nederland 7	

A


30 cm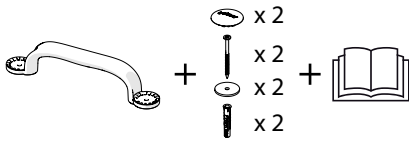

60 cm

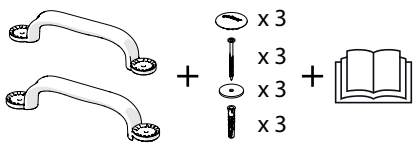B

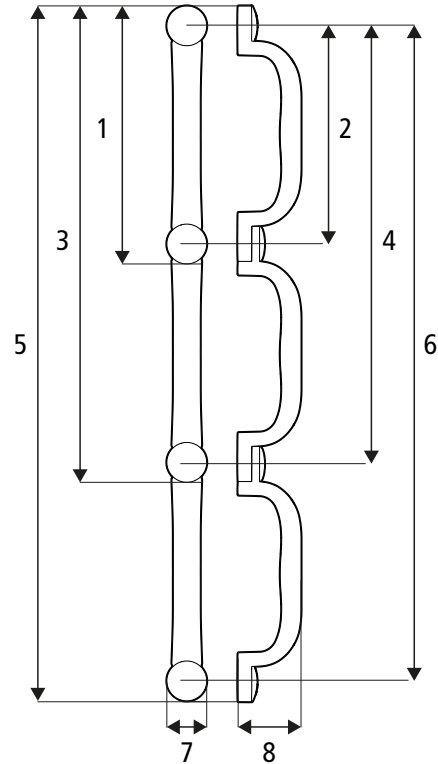B1: 355 mm (14")	B2: 300 mm (11¾")
B3: 655 mm (25¾")	B4: 600 mm (23½")
B5: 955 mm (37½")	B6: 900 mm (35½")
B7: 55 mm (2")	B8: 83 mm (3¼")

100 kg
(220 lbs)

>PP GF<




0,35 kg
(0,77 lbs)





C

1	2	3	4	5

D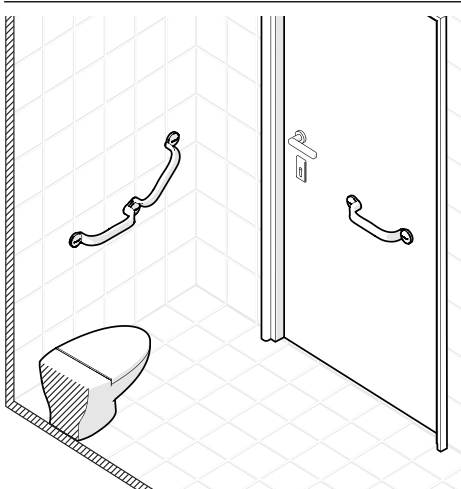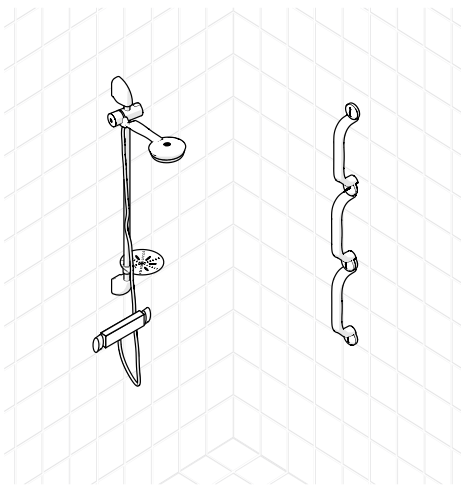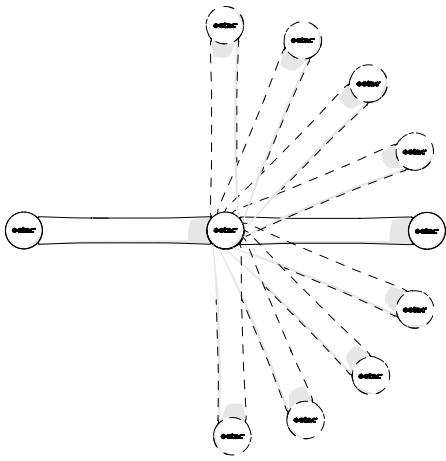

E

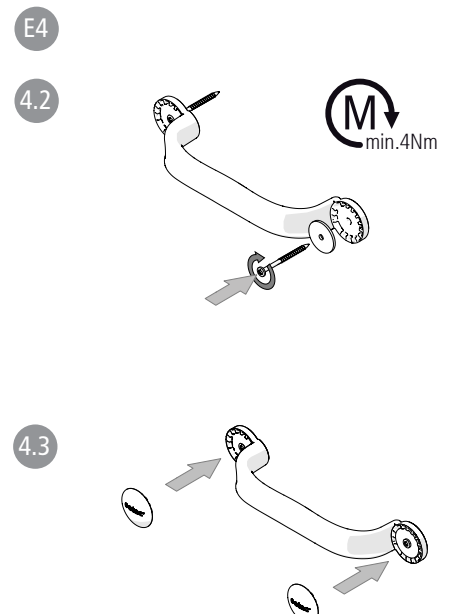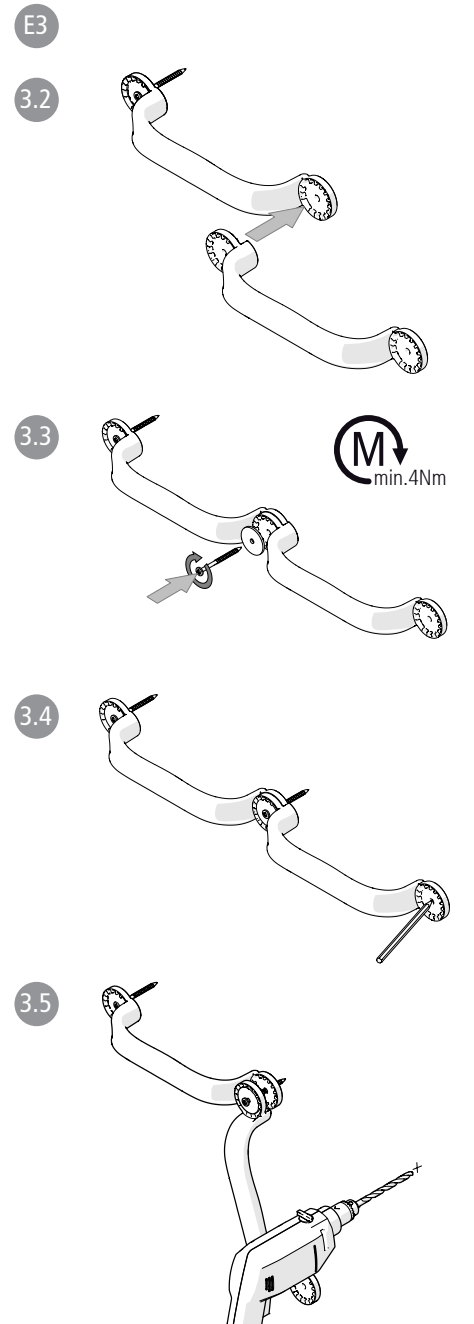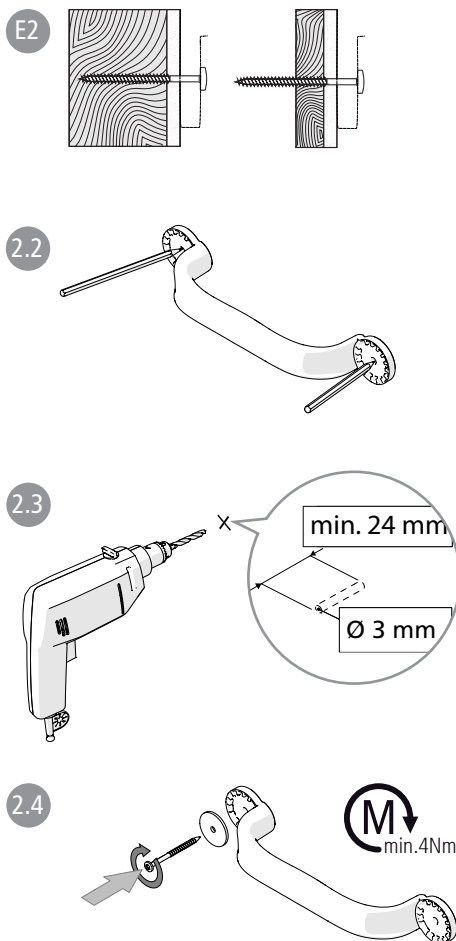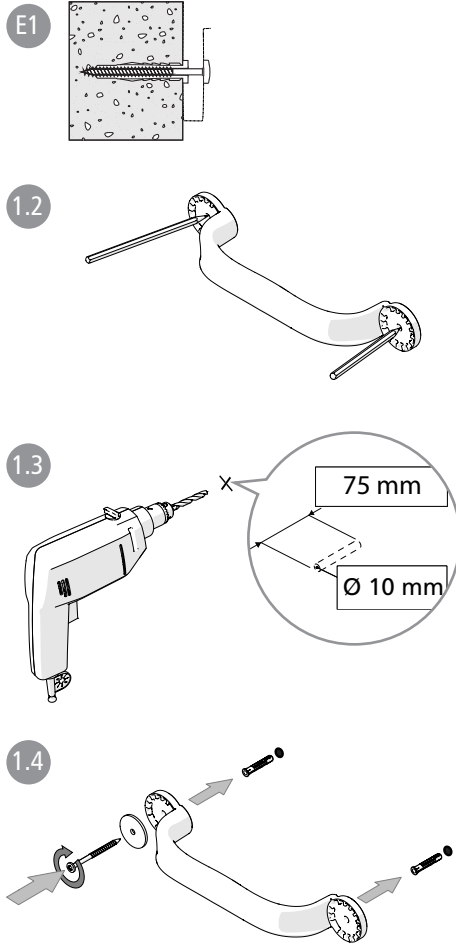**Svenska**

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en

English

Thank you for choosing an Etac product. In order to avoid damage during assembly, handling and use, it is important to read this manual and save it for future reference. You can also find it at www.etac.com. You can select your language via the "International" and "Local websites" link. Here you will also find other product documentation, such as prescriber information, pre purchase guide, and reconditioning instructions.

Intended use

Etac Flex is a wall-mounted support handle intended for use as a transfer and standing support for people with impaired mobility. Etac Flex is intended for use in home environments and in institutions, i.e. hospitals and nursing homes.

Etac Flex is not intended to be sat on, or to be used in any other way or in any other environment than those stated above.

Etac Flex has been tested and complies with the requirements set out in EN ISO 10993-1 and SS EN 12182.

The product conforms to the requirements of the Medical Devices Regulation (EU) 2017/745.

This manual applies for Etac Flex when mounting it with screws. When mounting it with glue, see separate manual under www.etac.com.

Contra indications

There are no known contraindications.

Content of the delivery [Figure A](#)

Technical data [Figure B](#)

This product should be recycled in accordance with national regulations. See material specifications for sorting ([figure B](#)).

Maintenance and safety [Figure C](#)

C1. Clean/disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution. Rinse and dry. The product requires no other maintenance.

C2. The product can be safely cleaned in a cabinet washer-disinfector at 85°C for 3 minutes.

C3. Max. user weight 100 kg (220 lbs).

C4. Check the screw fastener. A defective or incorrectly mounted product should not be used. In the event of a fault, contact your prescriber or retailer.
C5. Heated products can cause burns.

Constituent materials are corrosion-resistant.

Constituent materials are resistant to standard disinfection agents.

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

Guarantee, service life

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

Service life 10 years. For complete information regarding the service life of the product, see www.etac.com.

The product's date of manufacture can be read from the barcode on the product. The number 11 is shown below the barcode in brackets. The number combina-

tion after these brackets is the date of manufacture.

Storage

The product should be stored indoors in a dry place at a temperature above 5°C. If the product has been stored for a long time (more than four months), its function must be checked by an expert before use.

Positioning, use [Figure D](#)

Etac Flex can be mounted in different positions. The additional handle can be mounted at angles of 22.5° between 0 and 90°. Etac Flex can be extended by an unlimited number of additional handles.

Mounting options

Etac Flex has been designed to withstand a maximum user weight of 100 kg (220 lbs) and should only be mounted onto a wall with sufficient load-bearing capacity. Ensure that you use suitable fixtures.

Screws: Etac recommends the accompanying screw set for mounting on wood/plywood or concrete walls.

If in doubt about the wall's load-bearing capacity or the suitability of the fixtures, an assessment should be carried out by a qualified person.

In the case of other wall materials, or if the load-bearing capacity of the wall is deemed insufficient for any other reason, the wall can be reinforced by a qualified person.

Mounting on a wall with insufficient load-bearing capacity or unsuitable fixtures poses a danger to the user!

Mounting with screws [Figure E](#)

• Follow the instructions under sections **E1 & E4** for mounting on uncracked concrete C20/25 to C50/60. Always mount using Fischer UX R 10 plugs.

• Follow the instructions under sections **E2 & E4** for mounting on wood or plywood (min. 12 mm) with partition wall (min. 12 mm).

• Follow the instructions under sections **E1 or E2 & E3-E4** for mounting of Flex 60.

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Etac Supply Center AB